

DermulceraTM OINTMENT WOUND DRESSING

Rx Only

DESCRIPTION: DERMULCERATM OINTMENT contains Balsam Peru, Castor Oil USP, Glyceryl Monostearate and White Petrolatum.

ACTION: DERMULCERATM OINTMENT helps to deodorize and protectively cover pressure wounds (ulcers). DERMULCERATM OINTMENT provides a moist wound environment. The moisturizing properties of DERMULCERATM OINTMENT may also help in the prevention of cracking around the edges of sores.

INDICATIONS: DERMULCERATM OINTMENT is a wound dressing for topical use in the management of chronic and acute wounds, and dermal ulcers including: pressure ulcers (Stage I-IV), venous stasis ulcers, diabetic ulcers, first and second degree burns, surgical wounds, traumatic wounds, superficial wounds, ulcers resulting from arterial insufficiency and grafted wound/donor sites.

CONTRAINDICATIONS: DERMULCERATM OINTMENT is contraindicated in persons who have shown hypersensitivity to Balsam Peru, Petrolatum or any of the other ingredients used in this ointment.

USES: DERMULCERATM OINTMENT is easy to apply and quickly reduces odors frequently accompanying a decubitus ulcer. The wound may be left open or appropriate dressing applied. Please note that wounds generally heal poorly in the presence of hemoglobin or zinc deficiency.

WARNING: FOR EXTERNAL USE ONLY. Do not apply to fresh arterial clots. Avoid contact with eyes. Keep this and all other medications out of the reach of children. Keep tightly closed. Use only as directed by a physician. When applied to a sensitive area, a temporary stinging may occur.

USUAL DOSAGE: Apply a thin film of **DERMULCERA™ OINTMENT** topically a minimum of twice daily or as often as necessary. Wound may be left unbandaged or appropriate dressing can be applied. To remove, wash gently with an appropriate wound cleanser.

HOW SUPPLIED: **DERMULCERA™ OINTMENT** is supplied as follows:

<u>SIZE</u>	<u>NDC NUMBER</u>
60 GRAM TUBE	69437-502-60

STORAGE: Store at 20°C to 25°C (68°F to 77°F), excursions permitted between 15°C and 30°C (between 59°F and 86°F). Brief exposure to temperatures up to 40°C (104°F) may be tolerated provided the mean kinetic temperature does not exceed 25°C (77°F); however, such exposure should be minimized. [See USP Controlled Room Temperature]. Protect from freezing. See crimp for lot number and expiration date.

Distributed by:



Manufactured for:
Canton Laboratories, LLC
1015 Nine North Drive, Suite 200
Alpharetta, GA 30004

Customer Service
Telephone: (678) 867-2900
Fax: (770) 754-9850